


Help Articles


Denticon and Dentiray Capture – How to Take an X-ray

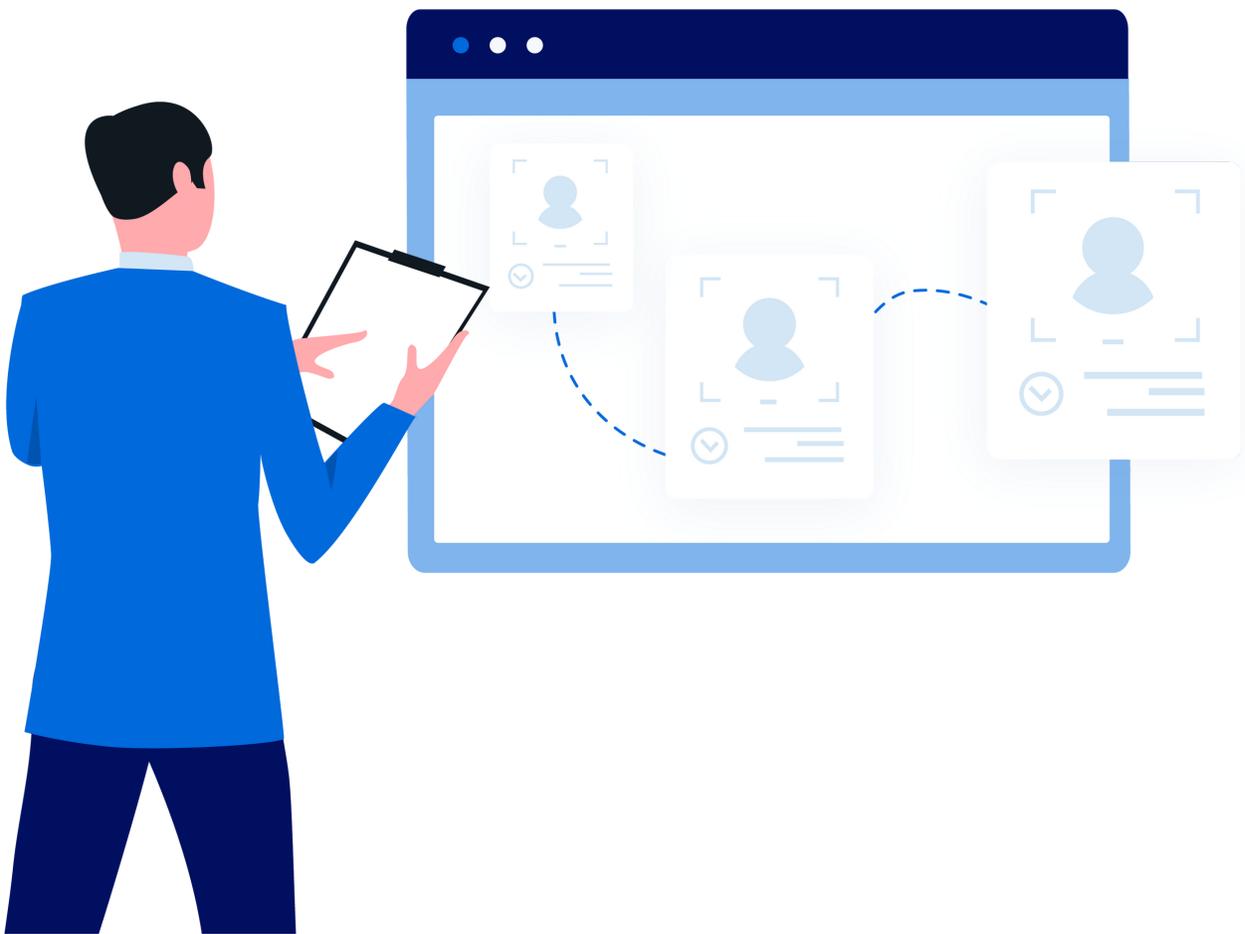

Denticon and Dentiray Capture – How to take an X-ray

This guide will show you the basic steps to bridge from Denticon into Dentiray Capture and take or capture an X-Ray.

1. Log into Denticon
2. Select a patient
3. Bridge using Launch X-ray System button

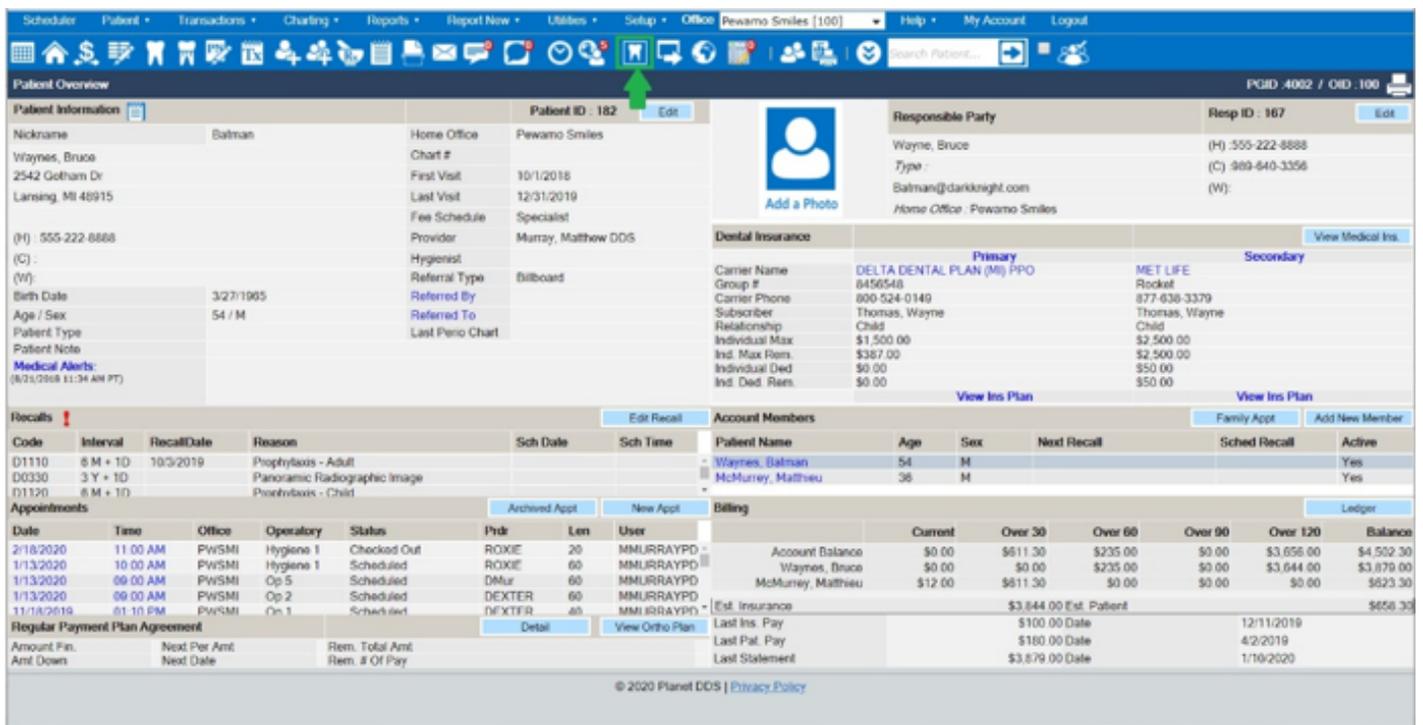

The screenshot displays the Denticon software interface for a patient named Batman. The interface includes a top navigation bar with various icons, a patient overview section with personal and dental insurance details, and a recalls table. A green arrow points to a flashing orange icon on the taskbar, which is the Dentiray Capture icon. The patient information section shows details such as Nickname (Batman), Home Office (Pewamo Smiles), and Birth Date (3/27/1905). The recalls table lists several scheduled appointments, including prophylaxis and panoramic radiographic images.

| Code | Interval | RecallDate | Reason | Sch Date | Sch Time |
|-------|----------|------------|------------------------------|----------|----------|
| D1110 | 6 M + 1D | 10/3/2019 | Prophylaxis - Adult | | |
| D0330 | 3 Y + 1D | | Panoramic Radiographic Image | | |
| D1120 | 6 M + 1D | | Prophylaxis - Child | | |

| Date | Time | Office | Operatory | Status | Prdr | Len | User |
|------------|----------|--------|-----------|-------------|--------|-----|-----------|
| 2/18/2020 | 11:00 AM | PWSMI | Hygiene 1 | Checked Out | ROXIE | 20 | MMURRAYPD |
| 1/13/2020 | 10:00 AM | PWSMI | Hygiene 1 | Scheduled | ROXIE | 60 | MMURRAYPD |
| 1/13/2020 | 09:00 AM | PWSMI | Op 5 | Scheduled | DMur | 60 | MMURRAYPD |
| 1/13/2020 | 09:00 AM | PWSMI | Op 2 | Scheduled | DEXTER | 60 | MMURRAYPD |
| 11/10/2019 | 01:10 PM | PWSMI | Op 1 | Scheduled | DEXTER | 60 | MMURRAYPD |

3A. If you are using Google Chrome, you'll need to locate the flashing orange Dentiray Capture icon on the taskbar and click it to open Dentiray Capture.

Patient middle initial:
Patient SSN: XXX-XX-4444
Patient birth date: 3/27/1965
Patient sex: M
Patient Phone: 5552228888
Patient provider: 100
Patient referring provider:

© 2020 Planet DDS | [Privacy Policy](#)



4. On the Main Menu, select Capture Images




5. Verify the patient information is the intended patient and select Capture Images on the following screen.




6. On the next screen, select the Imaging Device you are wanting to take the x-ray with. Imaging devices are typically listed by the manufacturer's name: I.E. Gendex, Schick, Carestream, etc. Some devices utilize a TWAIN interface, and most Intraoral cameras utilize the Direct Video interface.



7. The next screen will ask you to choose a tooth or a progression. You can select one or more teeth on the odontogram or you can choose a progression also referred to as a template or mount for a fixed series of images such as a Full Mouth Series or Bitewing series. Please note you can scroll through the list of Progressions to see all that are available. This is essentially where you're telling the software what you are taking the image of.

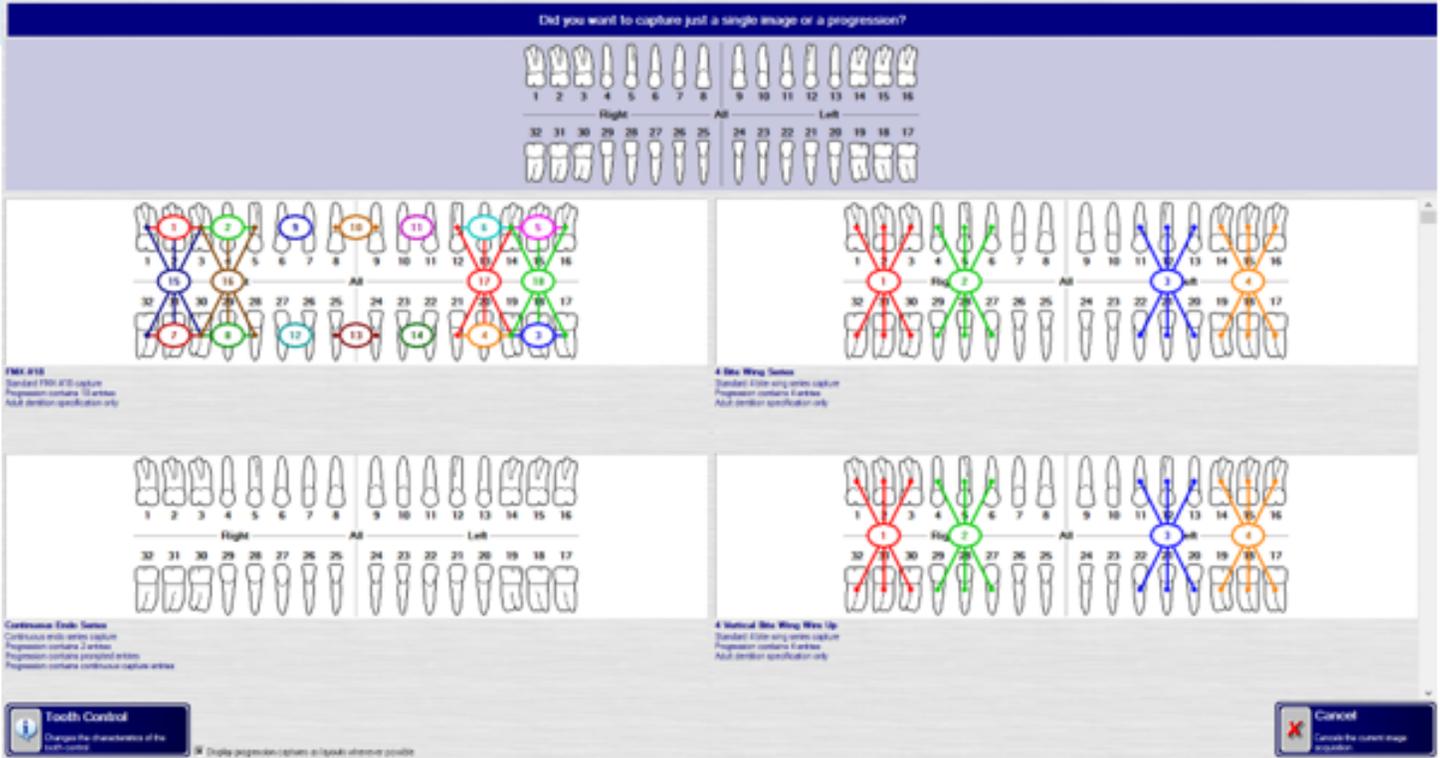


8. After you choose a tooth or a progression you be taken to the Capture screen. If you chose a tooth on the odontogram the sensor will attempt to go ready. If you selected a progression, you'll see the progression. To start taking images, select Start Progression Capture.

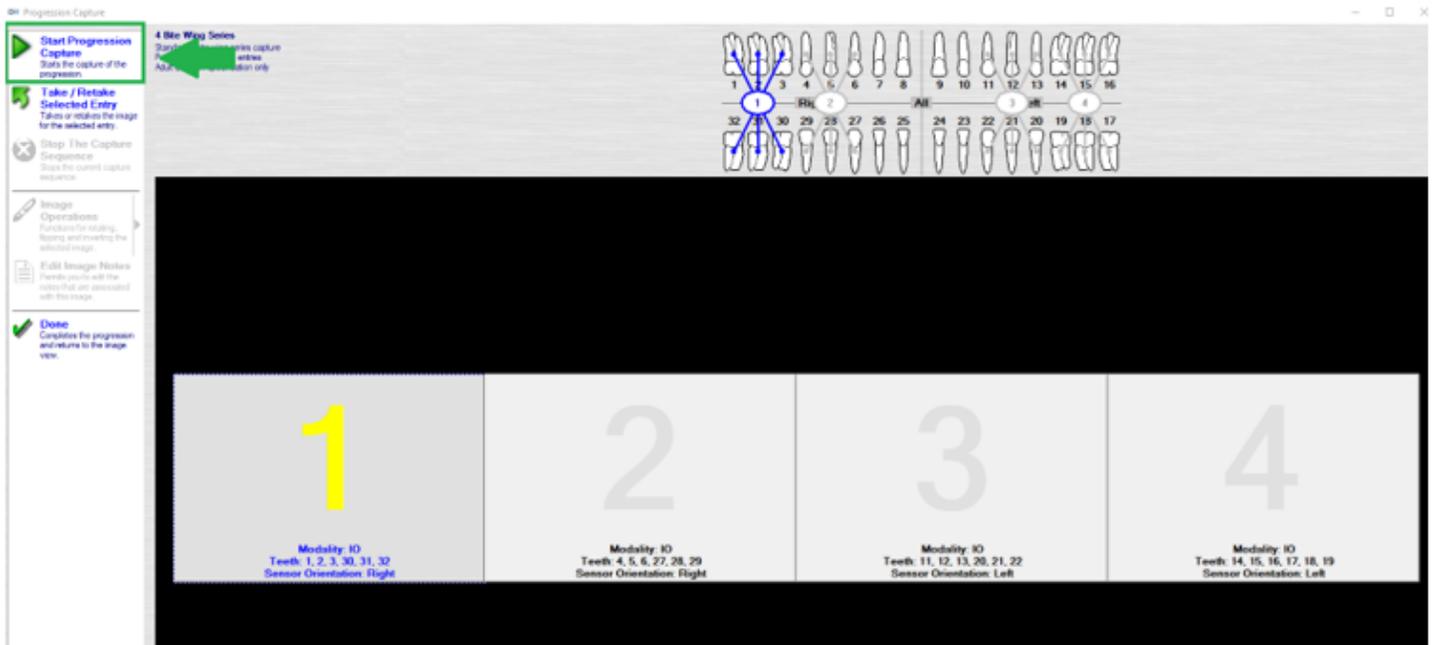

The Progression will automatically progress from shot to shot. All you need to do once the Progression is started is take the images.

9. When you have captured all of the images, select Done to return to the image viewer.

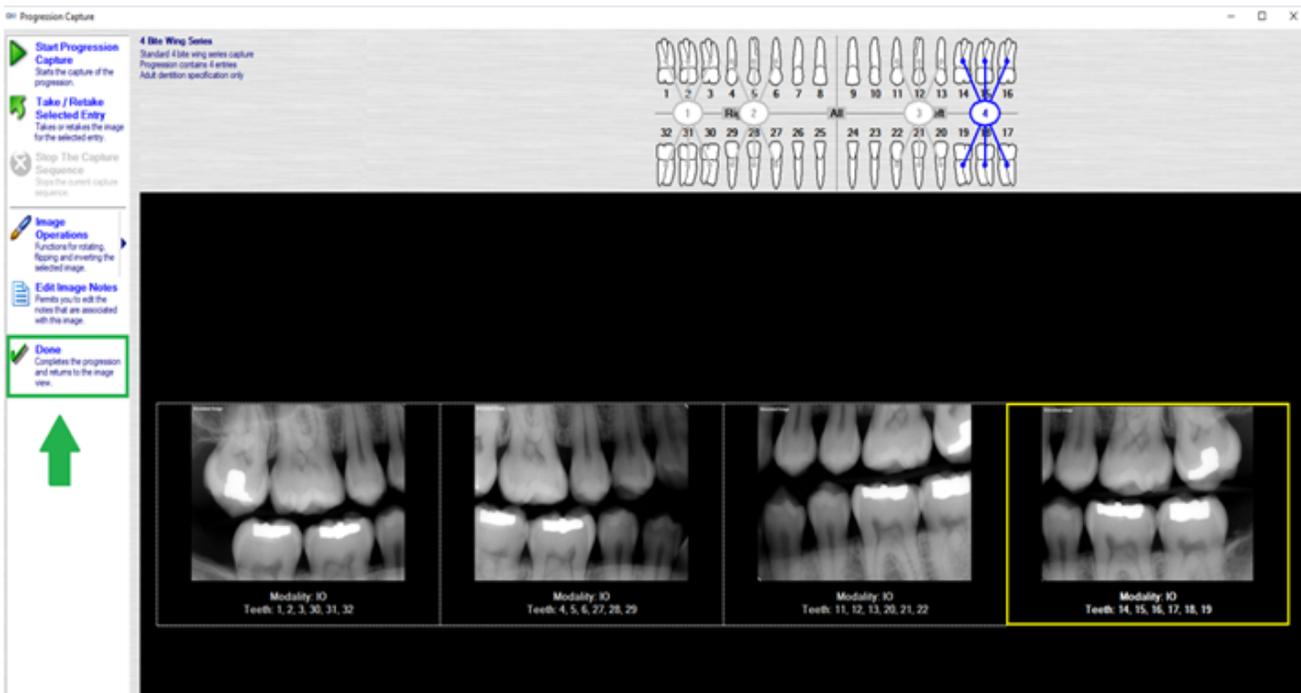

10. Once the Forwarding Status shows Idle in the Viewer, you may select Save & Go Back and then close the program to be ready for bridging the next patient. Those images will then also be available for viewing within Denticon's Dentiray Web.

