


Help Articles

How are we able to send/forward Apteryx XVWeb x-rays?

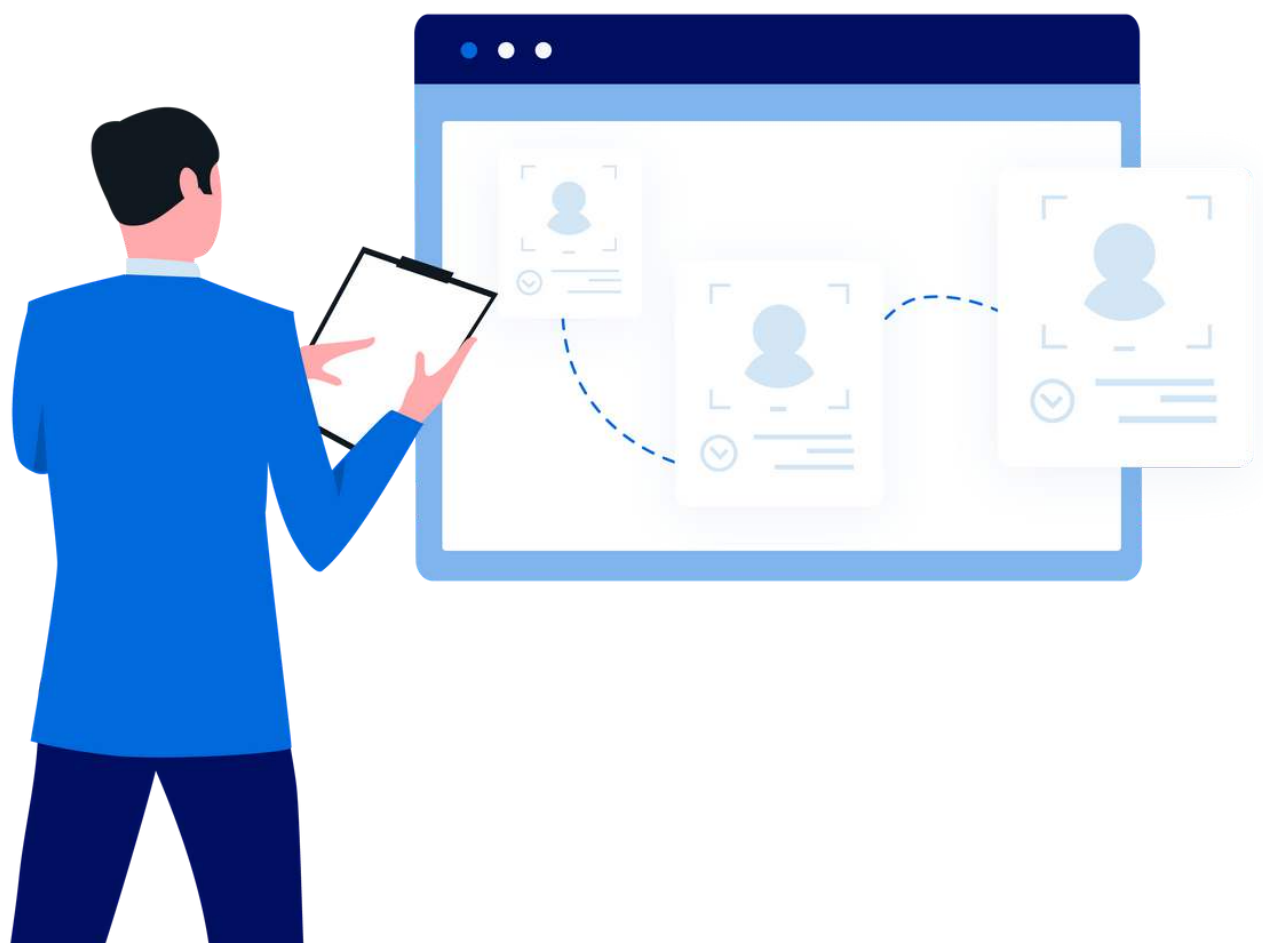

How are we able to send/forward Apteryx XVWeb X-rays?

To send x-rays:

1. Launch Apteryx XVWeb with the patient you want to send x-rays from. On the left column you can select which exam you want to send, then click on the Share button on the right.



2. Once you've selected the Share button you will have a few options to select from.

- Share Type: you can select User (permanent user) or Temp (Temporary) if Temp is selected you will need to fill a Username, email, allowing user to download the x-rays and a expiration date
- Share Content: Patient(entire patients x-rays) or Study(single exam)



3. Once you complete the selection press click on the Share button to send.

How to Video below.

<https://youtu.be/mlX3qBlqLME>

NOTE: For more information, please consult the full "XVCapture/XrayVision DCV User Manual" located within Denticon's Help Portal.